


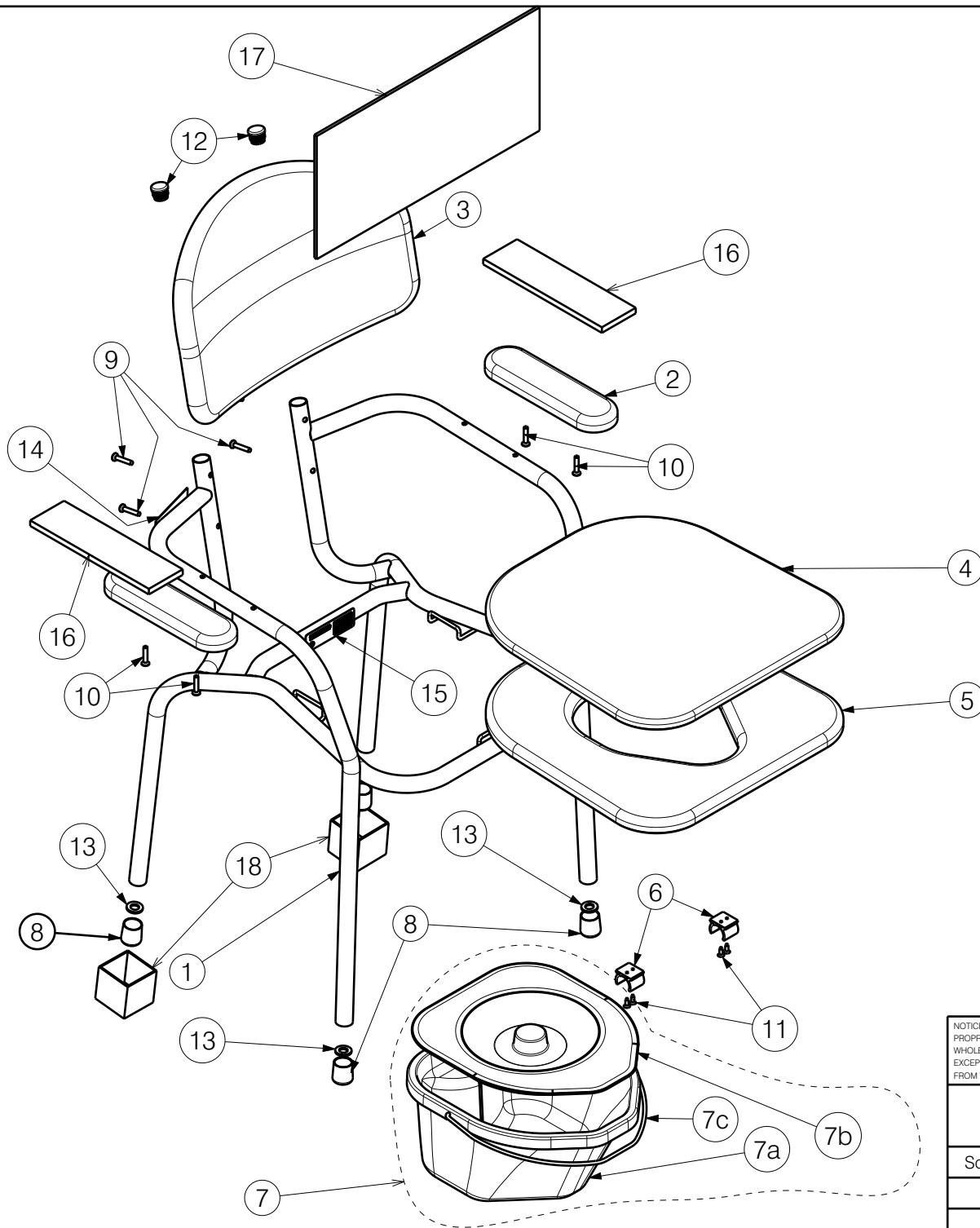

Glue N°8 into N°1
with Anobond 202
instant adhesive
(3900649)

*use for packaging, quantities for whole package of 5 chairs

Material			
No.	Plan	Article	Quantity
1	YP00376	18120...	1
2	---	2990076 (grey)	2
		2990079 (black)	
3	---	2990075 (grey)	1
		2990080 (black)	
4	---	2990077 (grey)	1
		2990081 (black)	
5	---	1998324 (grey)	1
		2990082 (black)	
6	XP00106	2990072	2
7	---	1996073	1
8	P00123578	1923642	4
9	---	1003903	4
10	---	1007954	4
11	---	1005767	4
12	---	1001608	2
13	---	1001561	4
14	Sticker	T001051	1
15	Sticker	T001097	1
16*	XP03128	2996685	8
17*	Cardboard	PP00961	4
18*	Feet	2994744	2

Bucket set - article 1996073

No.	Plan	Article	Quantity
7a	XP00097	2990065	1
7b	XP00098	2990066	1
7c	XP00113	GP00900	1

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.


E000120 Toliet Chair 9062

Scale: 1:8	Drafted	Verified	Confirmed
Date	11/02/2004	08-06-2026	08-06-2026
Initials	BS	MP	SD

Plan No.


E000120/R



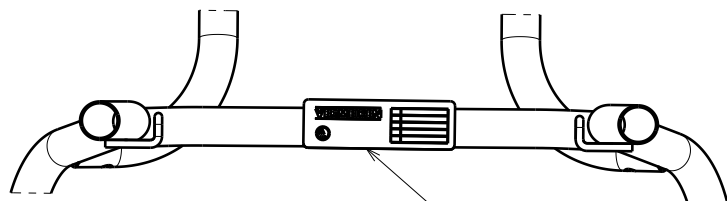

Central position

15

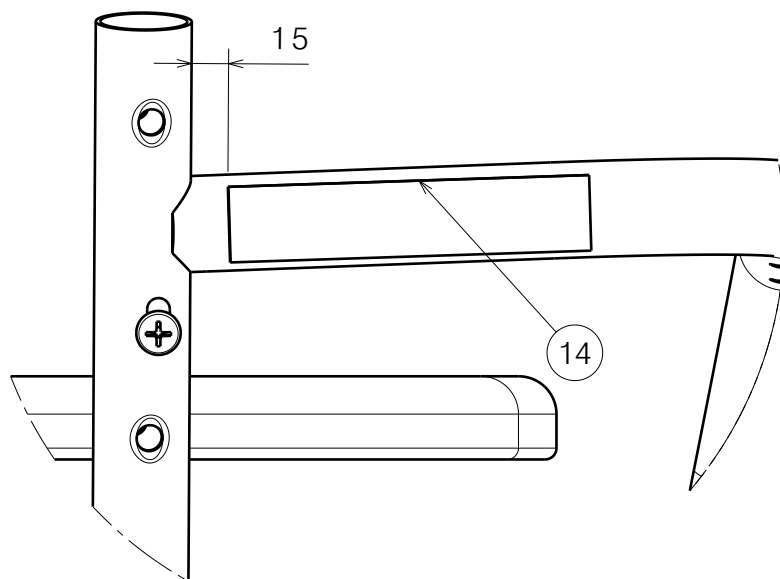

15

14

Material			
No.	Plan	Article	Quantity
1	YP00376	18120...	1
2	---	2990076 (grey)	2
	---	2990079 (black)	
3	---	2990075 (grey)	1
	---	2990080 (black)	
4	---	2990077 (grey)	1
	---	2990081 (black)	
5	---	1998324 (grey)	1
	---	2990082 (black)	
6	XP00106	2990072	2
7	---	1996073	1
8	P00123578	1923642	4
9	---	1003903	4
10	---	1007954	4
11	---	1005767	4
12	---	1001608	2
13	---	1001561	4
14	Sticker	T001051	1
15	Sticker	T001097	1
16*	XP03128	2996685	8
17*	Cardboard	PP00961	4
18*	Feet	2994744	2

*use for packaging, quantities for whole package of 5 chairs

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

E000120 Toliet Chair 9062

Plan No.

E000120/R

Scale: 1:1

Drafted

Verified

Confirmed

Date

11/02/2004

08-06-2026

08-06-2026

Initials

BS

MP

SD

