


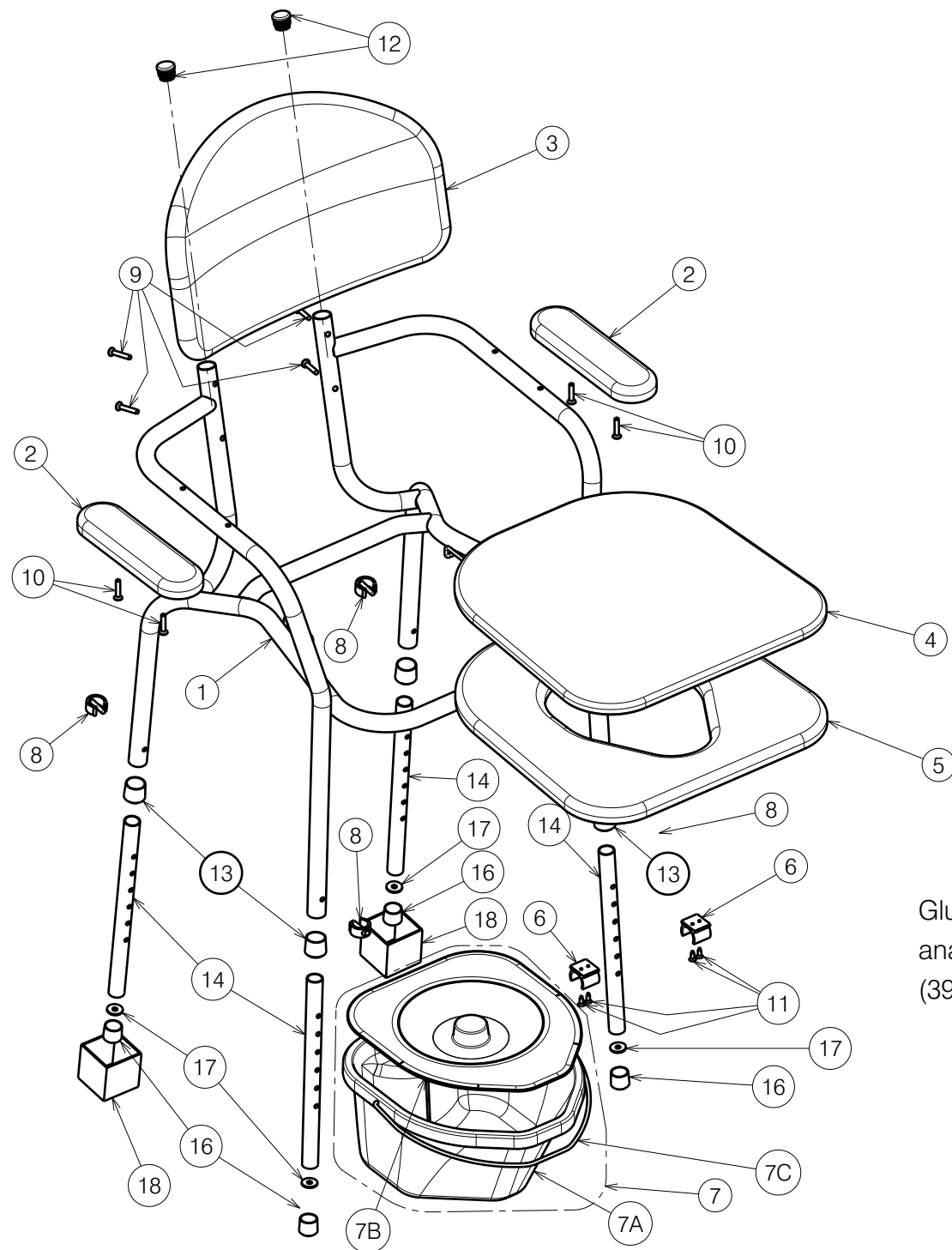

Material			
No	Plan N°	Article N°	Qty
1	YP00581	18155..	1
2	---	2990076 (grey)	2
	---	2990079 (black)	
3	---	2990075 (grey)	1
	---	2990080 (black)	
4	---	2990077 (grey)	1
	---	2990081 (black)	
5	---	1998324 (grey)	1
	---	2990082 (black)	
6	XP00106	2990072	2
7	---	1996073	1
8	XP00126	2990090	4
9	---	1003903	4
10	---	1007954	4
11	---	1005767	4
12	---	1001608	2
13	Y004353	F000411	4
14	YP00434	18135..	4
16	---	1001625	4
17	M7 DIN9021	1919656	4
18*	---	2994744	2

* Use for packaging, quantities are for stack of 5 chairs

RS3 Bucket set - article 1996073			
No	Plan N°	Article N°	Qty
7a	XP00097	2990065	1
7b	XP00098	2990066	1
7c	XP00113	GP00900	1

Glue N° 13 into N° 1 with
anabond 202 instant adhesive
(3900649)

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

TOILET CHAIR 9063				---
				Plan No.
				E000182/J
Scale: 1:9	Drafted	Verified	Confirmed	
Date	07/12/2004	08/06/2026	08/06/2026	
Initials	BS	MP	SD	